The HAMILTON-MR1 Transport kit allows you to securely transport and attach the ventilator to a trolley or to a shelf. It comprises the following components:

- Universal Mount handle, used to carry the device, as well as attach it to a standard rail
- Trolley/shelf mounting plate assembly, used to securely attach the device to a trolley and/or shelf

For setup, see the *HAMILTON-MR1 Transport Kit setup and installation instructions* (PN 624991). For details about positioning and working with the ventilator and the trolley, see the *HAMILTON-MR1 Operator's Manual*.

⚠ WARNING

- The ventilator may be attracted to the MRI scanner if it is placed too close to the scanner. Be sure to carefully position the ventilator using the TeslaSpy navigator as your guide.
- To prevent possible personal injury and equipment damage, make sure the ventilator is properly secured to the trolley or shelf.



The green lock indicator on the ventilator mounting plate (2) extends out from the plate when the top and bottom mounting plates are securely attached. It remains flush with the plate when the attachment is *not* secure.

- The ventilator must always be attached to a trolley or a shelf while in the MRI environment. Be sure the ventilator is screwed into the mounting unit, and if applicable, tethered to a wall anchor using the HAMILTON-MR1 tether.

1 Attaching the ventilator to a trolley or shelf

⚠ CAUTION

Ensure the trolley is locked in place before removing the ventilator.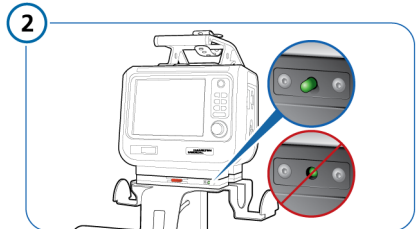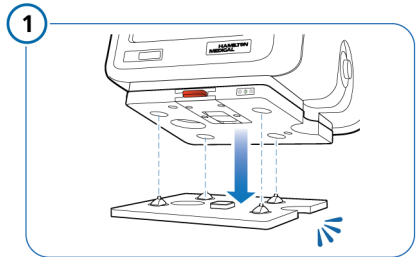

When attaching the ventilator to the plate, ensure it clicks into place. When securely attached, the green lock indicator extends out from the plate.

2 Detaching the ventilator from the trolley or shelf

To detach the ventilator from the trolley or shelf

1. Grab the handle.
2. Press the red **Release** button and pull the ventilator straight up off of the trolley or shelf.



3 Attaching the ventilator to a standard rail

⚠ WARNING

Always use the attachment strap to secure the ventilator when using the universal mount on a rail.

Using the Universal Mount, you can attach the ventilator to any standard rail.

Be sure the ventilator is securely attached to the rail and does *not* move freely on the rail during intrahospital transport.

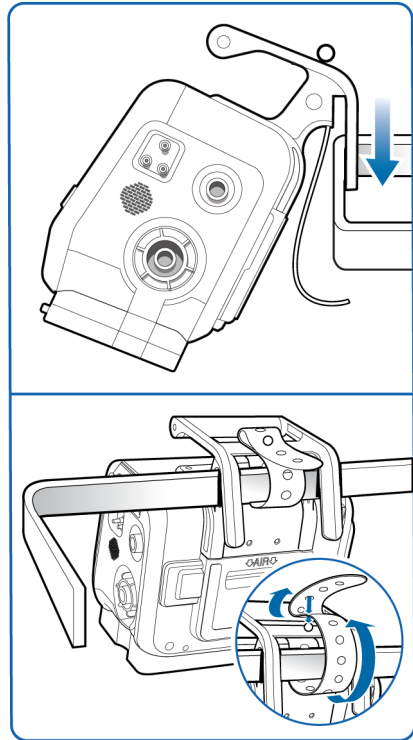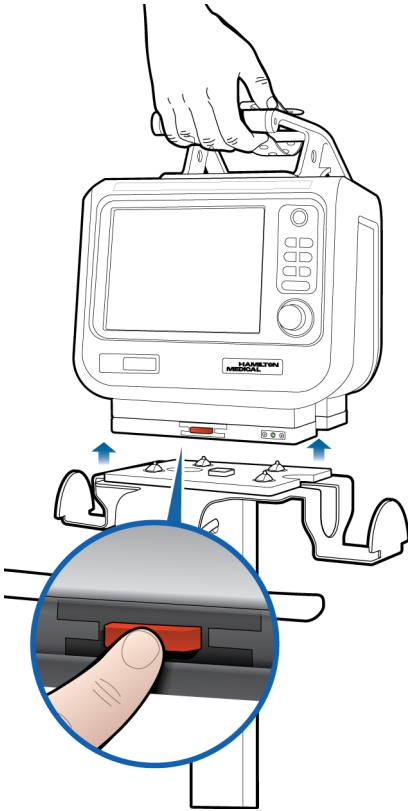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Hamilton Medical AG