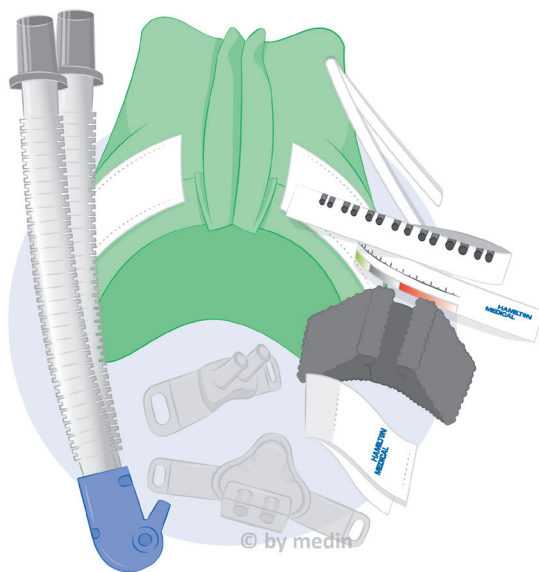
Instructions for use
Hamilton nCPAP Generator
and accessories
PN281976, PN281975,
PN282330

medin Medical Innovations GmbH
Adam-Geisler-Str. 1
82140 Olching, Germany

Distributed by:
Hamilton Medical
Via Crusch 8
7402 Bonaduz
Switzerland

*© Copyright 2021 – All contents, especially texts, photographs and graphics are copyright protected.
All rights including duplication, publication, editing and translation are reserved,
medin Medical Innovations GmbH*

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Warning: instructions for use and usage

Read these instructions for use carefully before using the products and follow the instructions precisely. The Hamilton nCPAP Generator may only be used by those with detailed knowledge of these instructions for use, which must be strictly observed. The Hamilton nCPAP Generator may only be used for the purpose described in the chapter "Intended use".

The Hamilton nCPAP Generator may only be used in combination with an appropriate HAMILTON ventilator, HAMILTON tubing set and appropriate patient monitoring.

The masks, prongs and bonnets may only be used in combination with a Hamilton nCPAP Generator and only with knowledge of these instructions for use.

The warnings and instructions must be followed.

The use of devices (e.g. ventilator and humidifier) which are used in combination with a Hamilton nCPAP Generator is described in the respective instructions for use of these devices. These instructions for use must also be followed.

1. Intended use**1.1. Hamilton nCPAP Generator**

The Hamilton nCPAP Generator is a single-use product for nCPAP therapy or non-invasive ventilation in neonates and premature infants in intensive care units. In such cases, the Hamilton nCPAP Generator serves as the patient interface. It is used in combination with a ventilator which controls the CPAP pressure and provides the Hamilton nCPAP Generator with the necessary gas.

1.2. Masks, Prongs and Bonnets

The bonnets, masks and prongs are products for use for nCPAP therapy or non-invasive ventilation of neonates and premature infants. They are to be used in combination with the Hamilton nCPAP Generator. The bonnets, which are available in different sizes, are used for fixation of the nCPAP generators, masks and prongs. The masks and prongs are single-use adapters which establish contact between the nCPAP generator and the patient. Their different sizes enable a connection to the patient's nose which is as gentle and leakproof as possible.

Warning: Use of the Hamilton nCPAP Generator

- Only for clinical use.
- The patient's blood oxygen saturation must be continuously monitored during use.
- The product may only be used by trained personnel.
- Hamilton nCPAP Generator, masks, single-use bonnets and prongs are single-use products and may be used on a single patient only. They may not be reused or reprocessed.
- The bonnets, masks and prongs should only be used in combination with a Hamilton nCPAP Generator.
- Hamilton nCPAP Generator may only be used in combination with a HAMILTON ventilator and HAMILTON tubing sets.

2. Functional principle and technical specification**2.1. Hamilton nCPAP Generator**

The Hamilton nCPAP Generator is a patient interface which can be used in CPAP mode or during non-invasive ventilation in combination with an appropriate Hamilton ventilator. The Hamilton nCPAP Generator can therefore be connected to the inspiratory and expiratory tube and thus to the ventilator. The CPAP parameters are adjusted and controlled on the ventilator. The ventilator also provides the necessary gas flow.

Inside the Hamilton nCPAP Generator, the expiratory and inspiratory gas flows are separated by a partition until they reach the prong adapter. The prong adapter can be swiveled and the direction of the prong adapter can be adjusted to the baby's size and nasal anatomy to simplify fixation.

A silicone prong or mask, chosen to suit to the baby's size, is connected to the prong adapter. The prong or the mask ensure gentle contact between the CPAP system and the baby's nose.

The Hamilton nCPAP Generator itself is fastened to a bonnet on the baby's head.

Technical data:

Expiration connector:	10 mm	Inspiration connector:	10 mm
Weight:	10 g	nCPAP pressure:	0-20 cmH ₂ O

2.2. Masks, Prongs, Bonnets

A divided fleece is sewn onto the upper side of the polyamide bonnets. A foam wedge (Minifoam for Hamilton nCPAP Generator) is placed between the fleece parts and the Hamilton nCPAP Generator is placed in this and secured with a separate hook and loop tape. The strips additionally included are used to stabilise the masks or prong position and, with their ends of hook and loop tape, they can be placed anywhere on the bonnet fabric.

The masks and prongs are made of 100% silicone. They all have loops for inserting the strips for fixation to the bonnet. In addition, they include a pair of holes for the connection to the Hamilton nCPAP Generator in which the swiveling adapter of the Hamilton nCPAP Generator is inserted for the connection to the Hamilton nCPAP Generator.

	Single-use bonnets	Masks	Prongs
Material	Polyamide	Silicone	Silicone
Use	Single use	Single use	Single use
	 		
Available sizes	XXS to XXXL	XS to XL	XS to XL

3. Set-up and application

Necessary equipment:

To administer effective CPAP therapy with the Hamilton nCPAP Generator, the following equipment is necessary:

- A Hamilton ventilator.
- An active respiratory gas humidifier.
- A Hamilton nCPAP Generator.
- An appropriate Hamilton tubing set.
- A bonnet that fits the baby.
- A mask or prong of the correct size
- Optional for ventilators which require external pressure measurement:
A T-piece for measuring the pressure (already contained in the medin tubing or can be ordered separately PN 260174, 260176).

Additional tools:

- A measuring tape to help select the sizes of bonnets, masks and prongs.

Warning: packaging

If the packaging of any part or the products themselves are damaged, these parts may not be used and must be discarded and disposed of.

Also check that the holes of the generator and of the prongs/masks are not blocked. In the event of blockage, the products in question must not be used.

Structure and set-up of the system (disposable bonnets)

- (1) Connect the tubing set to the ventilator and humidifier.
- (2) Connect the Hamilton nCPAP Generator to the inspiratory and expiratory tube. If your ventilator requires external pressure measurement, insert the T-piece between the end of the inspiration tube and the Hamilton nCPAP Generator for pressure measurement. The pressure measurement line can in turn be inserted into this and connected with the ventilator.
- (3) Close the prong adapter of the Hamilton nCPAP Generator by hand and adjust the CPAP parameters on the ventilator, e.g. the pressure. If this is not possible, check the tube connections to see if one is leaking. If all connections are correct and the Hamilton nCPAP Generator is causing the problems, the Hamilton nCPAP Generator must be discarded and may not be used.

Important instruction: connection and configuration

Detailed information on connecting and configuring the ventilation tubing system for nCPAP/nCPAP-PC with proximal pressure measurement (HAMILTON-C1/T1/MR1) and nCPAP-PS with distal flow sensor positioning (HAMILTON-G5/S1/C2/C3/C6) can be found in the operating manual of your ventilator.

Important instruction: defective Hamilton nCPAP Generator

If the Hamilton nCPAP Generator has visible damage, it may not be used.

- (4) Select a prong or a mask, using the measuring tape as an aid, if necessary. The prong should be selected such that it is large enough to seal the entire nostril. The mask should be selected such that it fits precisely along the sides of the nostrils. A prong is placed on the Hamilton nCPAP Generator such that the arched side of the prong points toward the patient's upper lip. The mask is attached such that it conforms to the shape of the nose.

Instruction: prongs

Always use the largest possible prong to avoid leaks and ensure the best possible positioning – the smaller the leaks are, the higher and more stable the nCPAP is.

Warning: prongs and masks

Always insert the Hamilton nCPAP Generator completely into the prongs or masks and don't use personal care products which can affect the positioning of the nCPAP interface near the prongs, masks or Hamilton nCPAP Generators, otherwise the prong or mask can slip off the Hamilton nCPAP Generator.

The prongs and masks should not be reprocessed and reused. They are for single use only. Reprocessed prongs and masks may harden and cause injury to the patient.

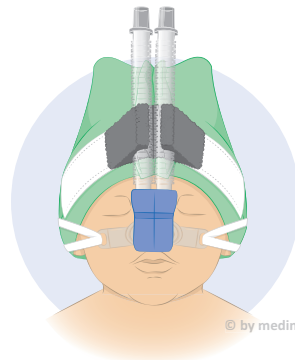
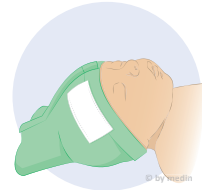
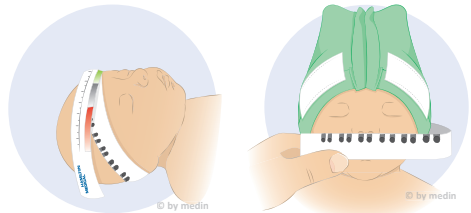
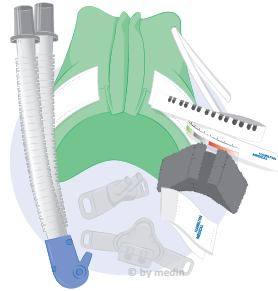
- (5.) Select a bonnet by means of the color coding and use the measuring tape as an aid, if necessary. The bonnet should be secured firmly enough but without leading to deformation of the head. Remove the separate hook and loop tape from the bonnet and put it aside, within easy reach. The bonnet is placed on the head such that the gap between the two fleece parts is centered over the forehead. The bonnet should cover the ears and be pulled down to the nape of the neck. Thread the fixation straps through the loops on the prong or on the mask (see picture).

Instruction: bonnets

Pull the bonnet down to just above the patient's eyebrows, as close as possible to the eyes – this makes it easier to secure the Hamilton nCPAP Generator.


Bonnets which are too small lead to deformation of the patient's head. Bonnets which are too large cannot ensure sufficient fixation of the Hamilton nCPAP Generator.

- (6) Select the appropriate angle for the prong adapter or the mask of the Hamilton nCPAP Generator (45° or 60°). If a prong is used, it generally yields the 45° angle; if a mask is used, it generally yields the 60° angle. Place the MiniFoam foam wedge between the two fleece pieces in the middle of the bonnet. Fit the Hamilton nCPAP Generator into the oval opening of the foam provided. Insert the prong into the nostrils. In doing so, maintain a distance between the nasal septum and the base of the prong. When using the mask, ensure that it rests evenly on the skin. The nostrils must be kept clear and should not be displaced by any part of the mask. Using the hook and loop tape, secure the Hamilton nCPAP Generator from above to the fleece pieces of the bonnet. Secure the hook and loop area of the straps in the area of the ears as an extension of the fixation loops of the prong. Do not pull the straps too tightly; they should only provide the lateral support needed.


Important instruction: prong and mask application

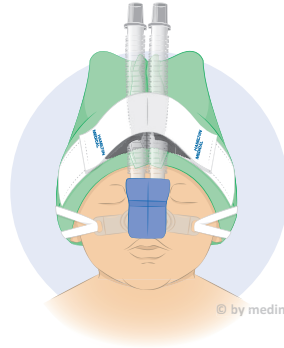- Do not insert the prongs fully into the nostrils. The connector block should not touch the nose.
- Regularly switching between prong and mask is an effective method for avoiding or reducing damage to the patient's skin.
- Do not pull the straps too tight. The Hamilton nCPAP Generator is held primarily by means of the bonnet and the foam. If the leak is too large, check the selected prong or mask size – pulling the straps more tightly does not by itself necessarily solve the problem.
- Excessively tightening the straps or inserting the prongs too far can lead to pressure points on the baby's nose or skin.

Important instruction: fixation of the inspiratory and expiratory tubes

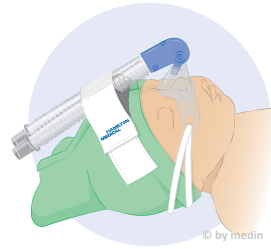- Ensure that the weight of the tubes does not pull directly on the patient's head or on the interface. Use cushioning to support the tubes.
- The inspiration and expiration tubing should slope away from the patient and the Hamilton nCPAP Generator so that the water condensation does not come into contact with the baby.

(7) Regularly check the CPAP parameters and the fixation.

Important instruction: regular check of the fixation

Check the fit and fixation of the Hamilton nCPAP Generator and the accessories regularly to avoid pressure points. Check the prongs and masks regularly for accumulation of secretions to avoid obstruction.

4. Ventilators which can be connected to the Hamilton nCPAP Generator

The Hamilton nCPAP Generator can be connected with any HAMILTON ventilator.

- HAMILTON-C6/S1/G5/C3/C2/C1/C1neo/T1/MR1

4.1. Masks, Prongs, Bonnets

All bonnets, masks and prongs can only be combined with the Hamilton nCPAP Generator.

5. Accessories

Hamilton nCPAP Generator 20 pieces, single use	PN 281976		
Minifoam 10 pieces, single use	PN 10064763		
Prongs - 10 pieces, single use			
x-small	PN 281977	large	PN 281980
small	PN 281978	large wide	PN 281982
medium	PN 281979	x-large	PN 281981
medium wide	PN 282328		
Masks - 10 pieces, single use			
x-small	PN 282745	medium	PN 281984
small	PN 281983	large	PN 281985
Masks - 5 pieces, single use			
x-large	PN 281933		
Bonnets - 10 pieces, single use			
xx-small	PN 282327	large	PN 281989
x-small	PN 281986	x-large	PN 281990
small	PN 281987	xx-large	PN 281991
medium	PN 281988	xxx-large	PN 281992
Breathing circuits - 15 sets - single use			
Single-use tubing sets, unheated, for Hamilton nCPAP Generator	PN 260169, PN 260170	Single-use tubing sets for Hamilton nCPAP Generator with humidifier chamber and heated inspiration and expiration	PN 260185

Warning: accessories

The Hamilton nCPAP Generator works only in combination with these masks, prongs and bonnets, otherwise the correct function of the device cannot be guaranteed. We recommend the use of the HAMILTON-H900 humidifier in order to provide the necessary humidification.

6. Storage and disposal

The Hamilton nCPAP Generator and its accessories should be stored in a dry, clean place. Temperatures between -30°C and 70°C have no negative effect on the Hamilton nCPAP Generator or accessories. The Hamilton nCPAP Generator and its accessories can be disposed of in the domestic waste.

EN

7. Symbols

The symbols explained below are used as labels on the product or in these instructions for use:

Symbol	Meaning	Symbol	Meaning
	CE mark		Temperature limit
	Medical device		Latex-free
	Follow instructions for use		Manufacturer
	Article number		Protect from sunlight
	Production lot number, batch		Date of manufacture
	Do not reuse		Use before
	Keep dry		Do not wash
	For use by medical prescription only		

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History of changes

Revision	Validity date	Changes
01	01.04.2018	First version of OEM Hamilton IFU.
02	08.06.2020	Alignment of page numbers, Change of name hook and loop fastener, inclusion of symbols, deleted reusable bonnets, wording adapted regarding MDE Standard, connected devices revised, word adapted regarding Measuring tape, Accessories revised
03	25.01.2021	Added PN281975, PN282330 on cover page Changes name into Instructions for use

Notes

Notes

Notes

Notes

medin Medical Innovations GmbH
Adam-Geisler-Str. 1
D- 82140 Olching

 +49 8142 448 460
info@medin-medical.com
www.medin-medical.com

For USA: RX only