

NIV masks fitting guide.

BiTrac full face mask




Why choose a BiTrac NIV mask for your patients?

- ✓ Comfortable and easy fit thanks to the silicone cushion¹
- ✓ Reduced risk of cross-contamination due to single patient use
- ✓ Different sizes for a simple mask-fitting process
- ✓ For adult and pediatric patients

¹ Sadeghi S. et al. Canadian Respiratory Journal Volume 2017.




Step 1: Use the sizing guide to select the correct mask size.

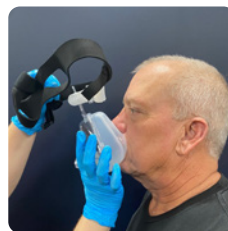

Step 2: Hold the head strap in front of the mask and position the bottom of the mask underneath the bottom lip.



Step 3: Flip the head strap over the patient's head and place the mask on the patient's face. Make sure the cushion is resting underneath the lower lip and on the bridge of the nose.

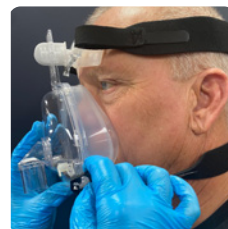

Step 4: Attach the bottom clips on the head strap to the mask. This should make an audible clicking sound.



Step 5: Adjust the top straps to fit. Do not overtighten the head straps.



Step 6: Once the top straps are secured, adjust the bottom velcro straps. Follow these steps again if the mask has been removed or when refitting the mask.

NIV masks fitting guide.

BiTrac MaxShield mask




Step 1: Use the sizing guide before selecting a mask. The mask should fit above the eyebrows and underneath the bottom lip.




Step 2: Unwrap the mask from the packaging. Hold the head strap in front of the mask and position the bottom of the mask underneath the bottom lip.



Step 3: Flip the head strap over the patient's head and place the mask on the patient's face. Make sure the cushion is resting underneath the lower lip and above the eyebrows.

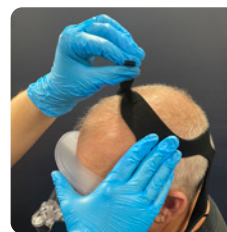


Step 4: Adjust the head strap on the crown of patient's head while holding the mask in place.

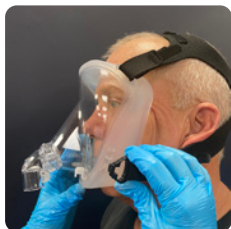

Step 5: Attach the two bottom clips on the head strap to the mask. This should make an audible clicking sound.



Step 6: Adjust the top straps to fit. Do not overtighten the head straps.

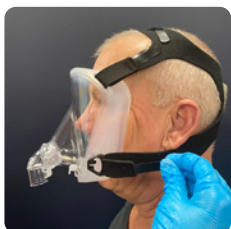

Step 7: Once the top straps are secured, adjust the bottom velcro straps.



Step 8: Follow these steps again if the mask has been removed or when refitting the mask.

Pulmodyne®

HAMILTON MEDICAL