Single-use Expiratory valve set adult/pediatric or neonatal for HAMILTON-C1/T1/MR1
 Expirationsventil für den Einmalgebrauch, Erwachsene/Pädiatrie oder Neonaten, für HAMILTON-C1/T1/MR1
 Système de valves expiratoires à usage unique adulte/enfant ou néonatal pour HAMILTON-C1/T1/MR1
 Conjunto de válvula espiratoria desechable para el HAMILTON-C1/T1/MR1, para pacientes adultos/pediátricos o neonatos
 Set valvola espiratoria monouso, adulto/pediatrico o neonatale per HAMILTON-C1/T1/MR1
 Conjunto de válvula expiratória de uso único adulto/pediátrico ou neonatal para HAMILTON-C1/T1/MR1
 适用于 HAMILTON-C1/T1/MR1 的一次性成人/儿童或新生儿呼气阀套件
 HAMILTON-C1/T1/MR1 用単回使用成人/小児または新生児用呼吸弁セット
 Одноразовый комплект клапана выдоха (педиатрический/взрослый или неонатальный) для аппаратов ИВЛ HAMILTON-C1/T1/MR1
 HAMILTON-C1/T1/MR1 için tek kullanımlık yetişkin/pediyatrik veya neonatal ekspiratuvar valf seti

Instructions for use | Gebrauchsanweisung | Instructions d'utilisation | Instrucciones de uso | Istruzioni per l'uso | Instruções de uso
 使用说明 | 使用説明書 | Инструкции по эксплуатации | Kullanım Talimatları

PN 624479/06 | 2021-12-15

Adult/pediatric **REF** 161186 Neonatal **REF** 161189

Consult the instructions for use. Gebrauchsanweisung beachten. Se référer aux instructions d'utilisation. Consulte las instrucciones de uso. Consultare le istruzioni per l'uso. Consulte as instruções de uso. 请参阅使用说明。使用説明書の説明に従ってください。См. инструкции по эксплуатации. Kullanım talimatlarına bakın.

Do not use if the packaging is damaged. Bei beschädigter Verpackung nicht verwenden. Ne pas utiliser si l'emballage est endommagé. No utilizar si el envase está dañado. Non utilizzare se la confezione è danneggiata. Não utilizar se a embalagem estiver danificada. 如果包装损坏, 请勿使用。梱包が破損している場合は使用しないでください。Не использовать при поврежденной упаковке. Ambalaj zarar görmüşse kullanmayın.

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EN Single-use Expiratory valve set adult/pediatric or neonatal for HAMILTON-C1/T1/MR1
 Instructions for use

WARNING:
 • Reprocessing of Hamilton Medical single-use products can affect the product properties and may cause injury to the patient. For example, a change to the surface structure during reprocessing may lead to a change in the tear strength or cause actual cracking. Furthermore, an altered surface structure may result in a microbial aggregation of spores, allergens and pyrogens, for example, or cause an increase in the number of particles released as a result of chemical changes in the material properties.
 • Hamilton Medical does not assume any liability for the proper functioning of expiratory valves if they are reprocessed and reused by the user.
 • Remove the protective cap before use: otherwise ventilation is not possible.
 • Mount the expiratory valve outside of the MR environment only! Otherwise the MRI scanner and the valve membrane may be severely damaged.

CAUTION:
 • Make sure the correct type of expiratory valve for your patient is installed:
 – Ensure the neonatal patient group is selected on the ventilator when using the neonatal expiratory valve.
 – You must use a neonatal expiratory valve for neonates.
 – Ensure the adult/pediatric patient group is selected on the ventilator when using the adult/pediatric expiratory valve.
 – You must use an adult/pediatric expiratory valve for adult pediatric patients.
 • Defective expiratory valves or expiratory valves that fail the tightness test must not be used.

NOTE:
 • Replace the expiratory valve set in accordance with hospital infection control procedures, or depending on the patient's secretions and nebulization of medication, if required.
 • Ensure you select the correct expiratory valve (adult/pediatric or neonatal) for your patient. If the expiratory valve type does not match the selected patient group on the ventilator, the Wrong expiratory valve alarm is generated.

- Follow internal hospital infection control procedures.
- Visually inspect the expiratory valve body, locking ring, and membrane. Discard the expiratory valve set if there is any sign of damage.
- Check the expiratory valve set prior to patient use. If the preoperational check fails, it may be repeated once. Discard the expiratory valve set if these preoperational checks fail.
- Any incident with the device leading to serious patient injury, death, or a potential threat to public health must be reported to the manufacturer and the relevant authorities.
- Ensure the membrane of the expiratory valve set is in its intended position in the ventilator.
- Read also the operating instructions of the HAMILTON-C1, HAMILTON-T1, and HAMILTON-MR1 ventilators.
- Ensure the membrane is aligned with the expiratory valve body and the metal faces up.

Intended use

Adult/Pediatric

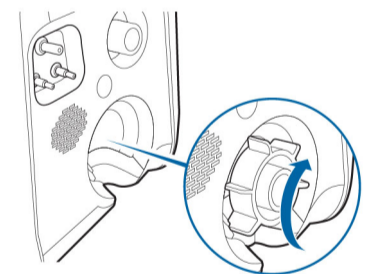
The Hamilton Medical single-use adult/pediatric expiratory valve set for HAMILTON-C1/T1/MR1 is an accessory for the exhalation gas path. Its main function is to control expiratory gas flow and maintain user-set pressure levels. The Hamilton Medical single-use expiratory valve set is not intended for use with equipment other than Hamilton Medical ventilators. The single-use expiratory valve set is intended for use by qualified, trained personnel.

Neonatal

The Hamilton Medical single-use neonatal expiratory valve set for HAMILTON-C1/T1/MR1 is an accessory for the exhalation gas path. Its main function is to control expiratory gas flow and maintain user-set pressure levels. The Hamilton Medical single-use expiratory valve set is not intended for use with equipment other than Hamilton Medical ventilators. The single-use expiratory valve set is intended for use by qualified, trained personnel.

Installing the expiratory valve set

The expiratory valve set must be mounted between the ventilator and the expiratory breathing circuit.

- Insert the expiratory valve into the exhaust port on the ventilator.
- Turn the expiratory valve set clockwise until you hear a "click".

Preoperational checks

Carry out a tightness test and a calibration of the flow sensor prior to operation. Follow the instructions as provided in the ventilator operator's manual.

Specifications

Operating temperature:	Adult/Pediatric: -15°C to 50°C (5°F to 122°F) Neonatal: -15°C to 40°C (5°F to 104°F)
Storage temperature:	-20°C to 60°C (-4°F to 160°F)
Storage, humidity:	5% to 95% relative humidity, noncondensing
Conical connectors:	OD22 /D15, according to ISO 5356-1:2015

Delivery

The expiratory valve set is delivered clean, ready for clinical use.

Disposal

A used expiratory valve set must be handled as a contaminated item. Follow all local, state, and federal regulations with respect to environmental protection when disposing of used expiratory valve sets.

For devices manufactured in Switzerland:

Manufacturer
 Hamilton Medical AG
 Via Crusch 8, 7402 Bonaduz, Switzerland
 ☎ +41 58 610 10 20
 info@hamilton-medical.com
 www.hamilton-medical.com

For devices manufactured and sold in the USA:

Printed in the USA
Manufactured for
 Hamilton Medical AG
 Via Crusch 8, 7402 Bonaduz, Switzerland
 ☎ +41 58 610 10 20
 info@hamilton-medical.com
 www.hamilton-medical.com

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