
BiTrac[®] Select[™]

BiTrac SE MaxShield Select[™]

Pediatric Mask with Standard Elbow 22mm Female
Interchangeable Elbow Compatible

bg	Инструкции за употреба	lv	Lietošanas norādījumi
cz	Návod k použití	nl	Gebruiksaanwijzing
da	Brugsanvisning	no	Bruksanvisning
de	Gebrauchsanweisung	pl	Sposób użycia
el	Οδηγίες χρήσης	pt	Indicações de Utilização
es	Instrucciones de uso	ro	Instrucţiuni de utilizare
et	Kasutusjuhend	ru	Инструкция по применению
fi	Käyttöohjeet	sk	Návod na použitie
fr	Mode d'emploi	sl	Navodila za uporabo
hr	Upute za uporabu	sr	Uputstvo za upotrebu
hu	Használati útmutató	sv	Bruksanvisning
it	Istruzioni per l'uso	tr	Kullanma Talimatları
lt	Naudojimo instrukcija		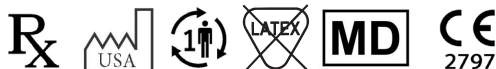

Intersurgical UAB
*Tel: + 370 387 66 611
e-mail: info@intersurgical.lt
Arnioniu g. 60, LT-18170, Pabradė, Lithuania

EC REP QNET BV
Kantstraat 19
NL-5076 NP Haaren
The Netherlands

Distributor in the USA:
Hamilton Medical, Inc.
201 Edison Way, Unit A
Reno, NV 89502-2305, USA
Distributed By:
Hamilton Medical AG,
Via Crusch 8, 7402
Bonaduz, Switzerland

Pulmodyne Inc.
2055 Executive Drive
Indianapolis, IN 46241 USA
www.Pulmodyne.com

Intended Use


The BiTrac SE MaxShield Select™ mask is an accessory for use with CPAP or Bilevel systems for ventilatory support.

Contraindications

- 1) This device should not be used on uncooperative or unresponsive patients.
- 2) This device may not be suitable for use on patients taking medication which may cause vomiting, have nocturnal vomiting, hiatal hernia or impaired cardiac sphincter function.
- 3) This mask may not be suitable for use on patients with the following conditions: glaucoma, recent eye surgery or dry eyes.

Warnings

- 1. Use only with a ventilator equipped with adequate alarms and safety systems for ventilator failure. This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
- 2. This mask does not have an integral leak port; therefore a separate exhalation device must be part of the circuit. Utilize the equipment manufacturer's recommended exhalation port in the designated circuit.
- 4. Any unusual chest discomfort, shortness of breath, stomach distension, belching, severe headache, drying eyes, eye pain, eye infections or blurred vision when receiving positive airway pressure or upon awakening should be reported to the clinician.
- 5. To minimize risks of vomiting during sleep, avoid eating or drinking for three (3) hours before use.
- 6. Make sure the mask and headgear are the correct size for the patient's face. If excessive discomfort is experienced, adjust the mask and head gear. If the discomfort continues, consider an alternative mask. The vent hole or holes associated with the mask should never be blocked.

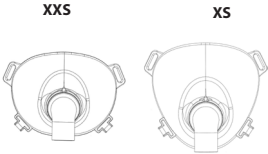Operating Specifications: 5°C to 40°C
at humidity range of 15% to 95%

Storage Specifications: -20°C to
60°C at relative humidity up to 95%
non-condensing

Achieving the Proper Fit:

- 1. Use the size guide to measure the size of the patients face.
- 2. The inside cushion should rest above the eyebrow and directly below the bottom lip. Refer to Figure 1.
- 3. When the correct size is achieved place the proper size mask on the patient.
- 4. Place mask on patients face, and pull the bonnet overhead.
- 5. Clip the mask into place and adjust the straps. Refer to Figure 2.
- 6. Continue with CPAP therapy per your protocols.

Use of Interchangeable Elbow Feature:

- 1. Remove the existing elbow from the mask by holding the adapter ring and twisting the elbow counter clockwise to disengage the bayonet lock (Refer to Figure B.)
- 2. Gently pull the elbow from the mask. (Refer to Figure C.)
- 3. Gently press the desired Interchangeable Compatible Elbow into the mask adapter ring and turn clockwise until fully seated. Do not use excessive force.

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