


O2 assist. Settings and ranges

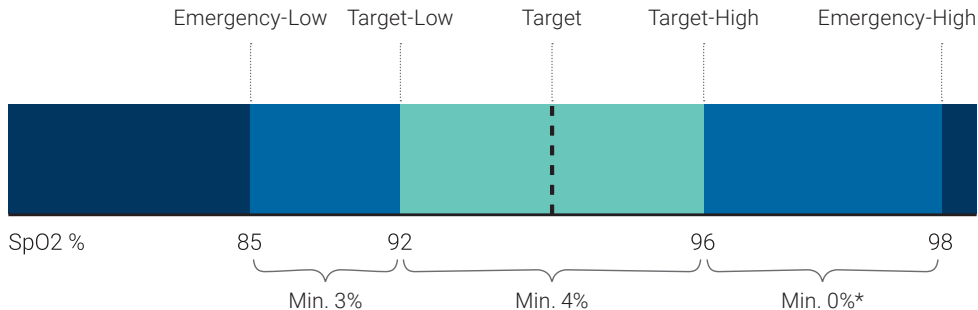


* Except when maximum values for Target High (99%) and Emergency High (100%) are set.

O2 assist working principles

Based on the defined settings, O2 assist adjusts the oxygen setting as follows:

No	User inputs	Description	Percent ¹	Time ²
1	SpO2 within target range	Setting is fine tuned to keep SpO2 in the middle of the target range		
2	SpO2 below target range	Increase	10%	30s
3	SpO2 above target range	Decrease	4%	60s
4	SpO2 below Emergency limit Low	Increase	50%	30s
5	SpO2 above Emergency limit High	Decrease	10 %	60s

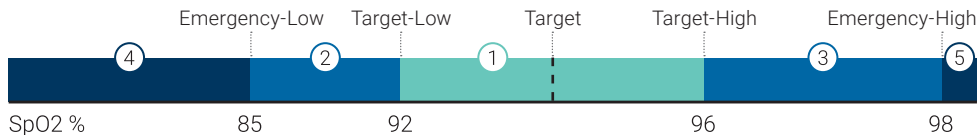

1 Change in % of the current oxygen setting | 2 Time between two changes