

# ASV<sup>®</sup> and INTELLiVENT<sup>®</sup>-ASV FAQs and troubleshooting

# Content overview

## Frequently asked questions (FAQs)

Ventilation settings	4
Safety window	13
Quick Wean	14

## Troubleshooting

Sensor-related issues	21
Ventilation settings	24

How to determine which specific condition to select?<sup>1</sup>

**Comments:** INTELLiVENT-ASV changes target ranges and initial settings depending on the set specific condition to make sure the ventilation settings are appropriate for the patient.

**Answer:**

- ✓ ARDS:  $\text{PaO}_2/\text{FiO}_2$  (P/F ratio)  $\leq 200$
- ✓ Chronic hypercapnia:  $\text{PaCO}_2 \geq 50$  mmHg in past medical history
- ✓ Brain injury: indication for strict control of  $\text{PaCO}_2$
- ✓ None: other cases

1. On some ventilators, the ARDS/Chronic hypercapnia/Brain injury conditions are listed under the heading *Specific conditions*; on other ventilators, the heading is *Patient conditions*.

### What to consider during a bronchoscopy?

**Comments:** **CAUTION!** Ventilation measurements and calculation of respiratory mechanics (**RCexp**) can be affected during bronchoscopy.

#### **Answer:**

- ✓ Change to PCV+ mode during bronchoscopy, decrease the sensitivity of the trigger, and then return to INTELLiVENT-ASV afterward.

### How to set %MinVol in ASV in passive patients?

#### Comments:

100% of %MinVol = 100 ml/kg/min (adult patient) is normal %MinVol (all of the following conditions fulfilled):

- ✓ Normocapnia
- ✓ At rest
- ✓ Normal metabolism
- ✓ Normal body temperature
- ✓ Normal lung function

#### Answer:

- ✓ If the PaCO<sub>2</sub> is higher or arterial pH is lower than your target, increase %MinVol.
- ✓ If the PaCO<sub>2</sub> is lower or arterial pH is higher than your target, decrease %MinVol.



Any lung disease would require %MinVol higher than 100% for normocapnia (due to increased physiologic dead space) and CO<sub>2</sub> production.

Why is the %MinVol limited to 200% in INTELLiVENT-ASV although I think it should be higher?

**Answer:**

- ✓ Check whether the patient really needs a %MinVol higher than 200% and whether this minute volume would be safe.
- ✓ Check that the patient height and sex settings are correct.
- ✓ If the patient is active, also check whether they are adequately sedated.
- ✓ If a higher %MinVol is really needed, set the %MinVol controller to manual; you can then increase %MinVol up to 350%.

### The PaCO<sub>2</sub> of my patient on INTELLiVENT-ASV is too high, or too low

#### Comments:

- ✓ The difference between PaCO<sub>2</sub> and PetCO<sub>2</sub> (referred as the CO<sub>2</sub> gradient) may be large in lung disease, but fortunately when PaCO<sub>2</sub> increases or decreases, PetCO<sub>2</sub> changes in the same direction.
- ✓ INTELLiVENT-ASV automatically adjusts the %MinVol on the basis of PetCO<sub>2</sub>, used as a surrogate of PaCO<sub>2</sub>.
- ✓ The PetCO<sub>2</sub> target range is automatically adjusted based on the peak pressure (automatic permissive hypercapnia), except if the patient condition is Brain injury.
- ✓ The PetCO<sub>2</sub> target range automatically proposed by INTELLiVENT-ASV can be adjusted by a manual control, the Target Shift.

#### Answer:

- ✓ If the desired PaCO<sub>2</sub> is significantly lower than the current value, apply a proportional shift of the PetCO<sub>2</sub> target range to the left, towards a lower level.
- ✓ If the desired PaCO<sub>2</sub> is significantly higher than the current value, apply a proportional shift of the PetCO<sub>2</sub> target range to the right, towards a higher level.

### The PaCO<sub>2</sub> of my patient on INTELLiVENT-ASV is too high, or too low



The **PetCO<sub>2</sub>** target can be shifted significantly ( $\pm 20$  mmHg – 2.6 kPa) to permit, if indicated:

- ✓ The manual setting of a CO<sub>2</sub> target considerably different from the normal value
- ✓ The manual compensation for large CO<sub>2</sub> gradients

When the **PetCO<sub>2</sub>** target is manually shifted to the left, it is possible that the desired drop in **PaCO<sub>2</sub>** is not completely reached, due to safety limitations involving permissive hypercapnia. Before attempting a further shift to the left, carefully consider which **PaCO<sub>2</sub>** level you would accept in this difficult-to-ventilate condition.

When the **PetCO<sub>2</sub>** target is manually shifted to the right, it is possible that the desired increase in **PaCO<sub>2</sub>** is not completely reached, due the safety limitations of minimum **%MinVol** equal to 70%. If a lower **%MinVol** is really needed, set the **%MinVol** controller to manual; you can then decrease **%MinVol** to below 70%.



### How to set the initial PetCO<sub>2</sub> target range?

**Comments:** The default **PetCO<sub>2</sub>** target range settings are appropriate for most cases. It is important, however, to always review the settings. Reasons for changing the target range include:

- ✓ The range is not suitable for a particular patient.
- ✓ There is a large CO<sub>2</sub> gradient.

For details, see the *INTELLiVENT-ASV Operator's Manual* for your ventilator.

#### Answer:

- ✓ Select one or more specific conditions only if the patient has a condition; if you are unsure, do NOT select any of the options.
- ✓ Use the default **PetCO<sub>2</sub>** target range to start with.
- ✓ Perform a blood gas analysis (BGA) after 30 minutes, or earlier if clinically indicated.
- ✓ If necessary, shift the **PetCO<sub>2</sub>** target range using the BGA values as a guide<sup>2</sup>.



On this page, the terms *target range* and *target shift* refer primarily to the **PetCO<sub>2</sub>** target range and the associated **Target Shift** control.<sup>3</sup>

2. Use the **Target Shift** control to adjust the target ranges. For details, see the *INTELLiVENT-ASV Operator's Manual* for your ventilator.

3. While not discussed very much here, you can also move (shift) the target range for **SpO<sub>2</sub>** to the left and to the right.

## How does ASV ventilate in spontaneously breathing patients?

**Comments:** ASV applies the principle of adapting pressure support for maintaining the patient's  $V_t$  at the  $V_t$  target.

The target **minute ventilation** is the minimum guaranteed by ASV, but the actual **minute ventilation** is determined by the patient.

- ✓ Pressure support is set by ASV to achieve the ASV target for  $V_t$ , while the patient is free to increase the respiratory rate above the target rate.
- ✓ ASV decreases the pressure support to a minimum of 5 cmH<sub>2</sub>O. When this level is reached, the patient's  $V_t$  may be higher than  $V_t$  target, but never lower.

### Answer:

Minute volume	$V_t$	$P_{INSP}$	RR	I/E
ASV + patient	ASV + patient	ASV	Patient	Patient

### How does INTELLiVENT-ASV ventilate in spontaneously breathing patients?

**Comments:** After five consecutive patient-triggered breaths with **PetCO<sub>2</sub>** below the upper **PetCO<sub>2</sub>** limit, the automatic adjustment of **%MinVol** also starts to take into account the patient's spontaneous **RR**.

When Brain injury is selected, the automatic adjustment of **%MinVol** is always based only on **PetCO<sub>2</sub>**.

**Answer:** In spontaneously breathing patients, the automatic adjustment of the **%MinVol** control depends on combined information from **PetCO<sub>2</sub>** and the spontaneous **RR**.

If **PetCO<sub>2</sub>** is below the upper limit (i.e., the patient is doing well), the adjustment of **%MinVol** depends on the spontaneous **RR**. If **RR** is too high, **%MinVol** and consequently pressure support are increased. Conversely if **RR** is low, **%MinVol** and consequently pressure support are decreased. Should **PetCO<sub>2</sub>** exceed the upper limit, the **%MinVol** adjustment will again be based only on **PetCO<sub>2</sub>**.

### How to ventilate a patient with high drive in ASV?

#### Comments:

Case 1: High need for support from the patient, frequently due to one or more of the following conditions:

- ✓ High O<sub>2</sub> consumption and CO<sub>2</sub> production, deteriorated lung function for gas exchange, metabolic acidosis

Case 2: Dysregulation of the respiratory center.

#### Answer:

Case 1: High drive can be controlled by combined interventions to lower the work of breathing and therefore oxygen consumption and CO<sub>2</sub> production:

- ✓ Increase in ventilatory support by increasing %MinVol
- ✓ Increased sedation and control of high temperature
- ✓ In the special case of metabolic acidosis, it is also necessary to manage the underlying metabolic derangement

Case 2: Sedation alone or sedation with paralysis can control high drive due to central dysregulation.

## How does INTELLiVENT-ASV ventilate a patient with high drive?

### Comments:

Case 1: High need for support from the patient, frequently due to one or more of the following conditions:

- ✓ High O<sub>2</sub> consumption and CO<sub>2</sub> production, deteriorated lung function for gas exchange, metabolic acidosis

Case 2: Dysregulation of the respiratory center

### Answer:


Case 1: High drive can be controlled by combined interventions to lower the WOB and therefore oxygen consumption and CO<sub>2</sub> production:

- ✓ Increased sedation and control of high temperature
- ✓ In the special case of metabolic acidosis, it is also necessary to manage the underlying metabolic derangement
- ✓ Normally INTELLiVENT-ASV automatically reacts by increasing the %MinVol control and therefore pressure support, thus helping to lower the high drive. If you consider the reaction of INTELLiVENT-ASV to be insufficient, you can switch to manual control of %MinVol and increase the setting

Case 2: Sedation alone or sedation with paralysis can control high drive due to central dysregulation

How does the shape of the ASV safety window change according to the patient's lung condition?

**Comments:** The shape of the safety window changes based on the patient's lung mechanics. The calculations are as follows:

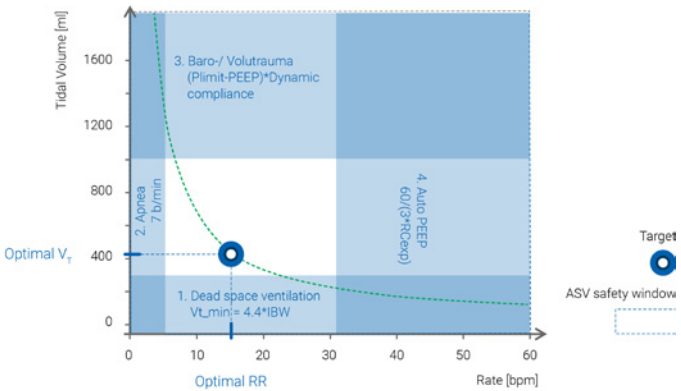

Answer:



Normal lungs: large square-shaped safety window




Restrictive lung diseases: wide, low rectangular-shaped safety window



Obstructive lung diseases: tall, narrow rectangular-shaped safety window

### What happens when Quick Wean is activated?

#### Comments:

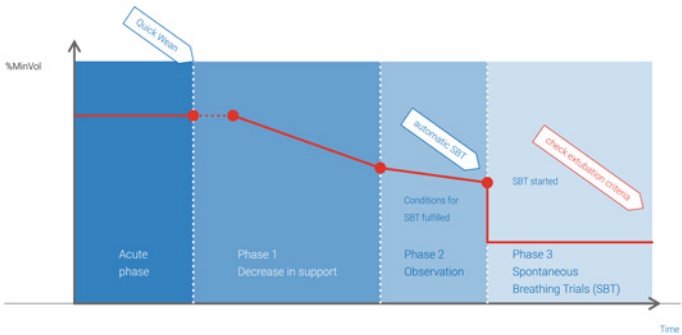

The following actions aid the de-escalation of ventilatory support:

- ✓ The target **PetCO<sub>2</sub>** is increased by 5 mmHg (0.6 kPa).
- ✓ The lower and upper target **RR** are increased.
- ✓ If the patient's condition is considered stable, **%MinVol** and therefore pressure support are gradually decreased.
- ✓ The readiness-to-wean criteria are screened.




Quick Wean is not available when **Brain injury** is selected.

When can/should activation of automated spontaneous breathing trials (SBTs) be considered?

**Comments:** Enable SBTs only to evaluate the patient for separation from the ventilator.

**Answer:** Consider SBTs when “non-respiratory” readiness-to-wean criteria are met:

- ✓ Patient is awake.
- ✓ Cough reflex is present.
- ✓ Hemodynamics are stable.
- ✓ Minimal vasopressor support is required.

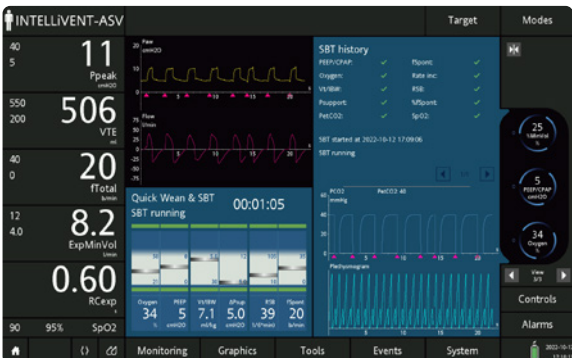



### Why do SBTs not start?

**Comments:** Possible causes:

- ✓ The patient is passive.
- ✓ %MinVol is set to Manual.
- ✓ Quick Wean and Automatic SBT are disabled.
- ✓ The criteria for starting SBTs are not met; the patient's dependency on the ventilator is too high to start an SBT.

**Answer:**

- ✓ Make sure the patient is breathing spontaneously and meets the criteria before enabling Quick Wean.
- ✓ Make sure Quick Wean and Automatic SBT are enabled.
- ✓ Check if %MinVol is set to Automatic.
- ✓ Reassess the patient for severity of respiratory failure and sedation.

When can/should enabling of Quick Wean (without automated SBTs) be considered?

**Comments:** Quick Wean (without automated SBTs) can be enabled even if the patient is still on low levels of sedation and/or low doses of vasopressors.

**Answer:** Consider enabling Quick Wean without automated SBTs when all of the following criteria are met:

- ✓ Sedation is stopped or decreased to “comfort sedation”.
- ✓ Hemodynamic condition is stable.
- ✓ Oxygenation is adequate.
- ✓ Patient is quiet and spontaneously breathing, and you have the clinical impression that they could do more.

How often can/should performing SBTs be considered?

### Comments:

#### First SBT was successful

Assess patient readiness for extubation, and continue with NIV, CPAP or high flow nasal cannula therapy, if necessary.

#### First SBT failed

- ✓ Identify the causes of the failure and the reasons why the patient still needs ventilatory support. Correct if possible.
- ✓ Try a second SBT after the causes of failure have been corrected, as long as the patient still meets the general weaning criteria.

**Answer:** If your organization's weaning protocol does not indicate otherwise, it is recommended not to do more than one SBT within a 24-hour period.<sup>4,5</sup>

4. Esteban, A. N Engl J Med. 1995 Feb 9;332(6):345-50.

5. MacIntyre, N. Chest. 2001 Dec;120(6 Suppl):375S-95S.

### How to limit the number of automated SBTs performed per day?

#### Comments:

- ✓ SBT can be manually started by touching **Start SBT**, provided that the patient is spontaneously breathing.
- ✓ Touch **Stop SBT** to stop an ongoing SBT.

#### Answer: When **Quick Wean** is enabled:

- ✓ You can disable automated SBTs and start an SBT manually, provided that the patient is spontaneously breathing.
- ✓ You can specify when automated SBTs can take place, by setting the **After** and **Before** times for the SBT time-range parameter.

To perform a single automated SBT:

#### HAMILTON-G5/S1

Set Time between two SBTs to --- (OFF)

- ✓ Only one SBT will be performed.

Set Time between two SBTs to 24 hours

- ✓ Only one SBT per day will be performed.

#### HAMILTON-C3/C6

Set Time between two SBTs to (30–240 min)

- ✓ If only one SBT is planned, deactivate automated SBTs after the first SBT is completed.

Can Quick Wean be enabled for difficult-to-wean patients?

**Answer:** Yes. For these patients, proceed as follows:

- ✓ Enable Quick Wean.
- ✓ Closely monitor the patient's breathing efforts and signs of fatigue.
- ✓ If you judge that the ventilatory support provided to your patient is not sufficient, disable Quick Wean to return to normal INTELLiVENT-ASV operation.

### Poor-quality SpO2 signal with frequent alarms

Oxygenation adjustment OFF (no SpO2)

#### Possible causes:

- ✓ Patient has low perfusion.
- ✓ NIBP cuff is on the same arm as the sensor.
- ✓ Sensor is incorrectly positioned.

#### Solutions:

- ✓ Try a different finger and change the position every 4 hours (reusable) or every 8 hours (single use).<sup>6</sup>
- ✓ Try an ear sensor.
- ✓ Temporarily set **Oxygen** and **PEEP** to **Manual**.<sup>7</sup> Try setting them to **Automatic** again once distal perfusion has improved.



6. Times according to the manufacturer's recommendation (Masimo and Nihon Kohden).

7. Setting a control to **Manual** can also be referred to as deactivating a controller; setting a control to **Automatic** can be referred to as activating a controller.


### Poor-quality PetCO<sub>2</sub> signal with frequent alarms

Ventilation adjustment OFF (no PetCO<sub>2</sub>)

**Possible causes:** Condensation or secretions may be present in the CO<sub>2</sub> airway adapter.

**Solutions:** Check the CO<sub>2</sub> sensor:

- ✓ If there is moisture in the adapter and the sensor is not properly positioned, correct the position (the circuit must be angled upwards, the adapter windows must be vertical).
- ✓ If there are secretions in the adapter, replace it and recalibrate the sensor offline.

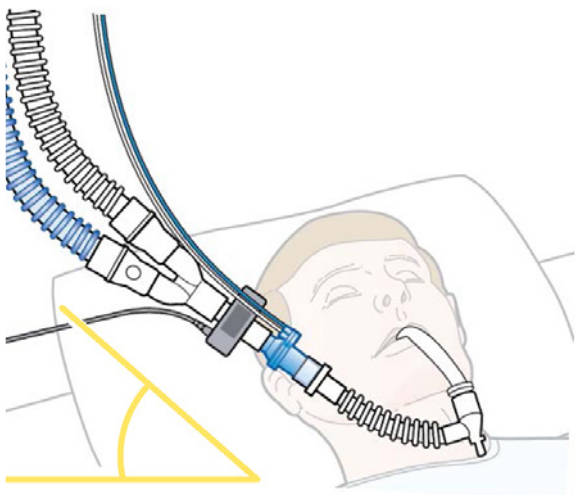

### The monitor for vital signs shows a different SpO<sub>2</sub> result than the ventilator

#### Possible causes:

- ✓ Manufacturers use different algorithms to measure SpO<sub>2</sub>.
- ✓ The sensors are placed too close to each other, causing signal interference.

#### Solutions:

If using multiple sensors, ensure there is at least one finger without an SpO<sub>2</sub> sensor between the sensors.

In doubt, perform a blood gas analysis (BGA) with SaO<sub>2</sub> measurement. Consider that:

- ✓ On average SpO<sub>2</sub> is an overestimation of SaO<sub>2</sub>, with minimal bias and good precision if SaO<sub>2</sub> is > 90% and a poorer match in the case of lower SaO<sub>2</sub> values.
- ✓ With abnormal levels of COHb or MetHb, the match between SpO<sub>2</sub> and SaO<sub>2</sub> is lost and conventional pulse oximetry is to be considered unreliable.

Whenever you consider the SpO<sub>2</sub> provided by the ventilator to be unreliable, set Oxygen and PEEP to Manual.



INTELLiVENT-ASV repeatedly applies high levels of FiO<sub>2</sub> for short periods of time

### Possible causes:

- ✓ Deterioration of the patient's condition
- ✓ Low SpO<sub>2</sub> sensor signal resulting in an inaccurate SpO<sub>2</sub> measurement
- ✓ Other factors causing a sudden drop in SpO<sub>2</sub> (disconnection, suctioning, patient positioning, and so on)
- ✓ PEEP is at high limit

### Solutions:

- ✓ Check the patient's condition.
- ✓ Check the SpO<sub>2</sub> signal quality and change the sensor positioning or use an ear sensor, if necessary.
- ✓ Increase the high PEEP limit, if clinically indicated.
- ✓ Set the Oxygen control to Manual, if the above is not desired.

### SpO<sub>2</sub> does not match with SaO<sub>2</sub> measured by CO-oximetry on a blood sample

#### Possible causes<sup>8</sup>:

- ✓ Poor perfusion
- ✓ Motion
- ✓ Skin pigmentation
- ✓ Nail polish
- ✓ Vascular dyes (for example, methylene blue)
- ✓ Severe anemia
- ✓ Abnormal levels of dyshemoglobins like Carboxyhemoglobin, Methemoglobin, Sulphemoglobin

#### Solutions:

- ✓ Assess the patient, verify proper sensor placement, take action to improve the signal quality.
- ✓ In the presence of abnormal levels of dyshemoglobins and in any case if the difference between SpO<sub>2</sub> and SaO<sub>2</sub> is > 4%, set Oxygen and PEEP to Manual.

RR is too high and %MinVol is at 200%

### Possible causes:

- ✓ Inappropriate setting of height resulting in an inappropriate IBW/PBW calculation
- ✓ High respiratory drive (tachypnea) due to non-respiratory causes such as metabolic acidosis, pain, anxiety, and so on

### Solutions:

- ✓ Ensure the patient's height is correctly set.
- ✓ If the tachypnea is not related to a respiratory issue, it will not respond to an increase in %MinVol. Treat non-respiratory causes for tachypnea.

### PEEP does not decrease

#### Possible causes:

- ✓ PEEP decreases only if SpO<sub>2</sub> is above the target range.
- ✓ Oxygenation has not yet improved sufficiently.
- ✓ The SpO<sub>2</sub> target has been shifted to supra-normal values.
- ✓ PEEP is at the lower PEEP limit.

#### Solutions:

- ✓ Check the SpO<sub>2</sub> target range, and adjust, if appropriate.<sup>2</sup>
- ✓ Decrease the lower PEEP limit.
- ✓ Decrease PEEP manually, if clinically indicated.

2. Use the **Target Shift** control to adjust the target ranges. For details, see the *INTELLiVENT-ASV Operator's Manual* for your ventilator.


$V_t$  is very low and RR is very high

### Possible causes:

- ✓ The minimum  $V_t$  is 4.4 ml/kg IBW/PBW for patients with severe restrictive lung disease.
- ✓  $P_{limit}^9$  might be too low.
- ✓  $RC_{exp}$  is very short.

### Solutions:

- ✓ Check  $P_{plat}$ .
- ✓ Check  $P_{limit}$  and increase it, if appropriate.
- ✓ Check for the cause of such a severe restrictive state and correct if possible.

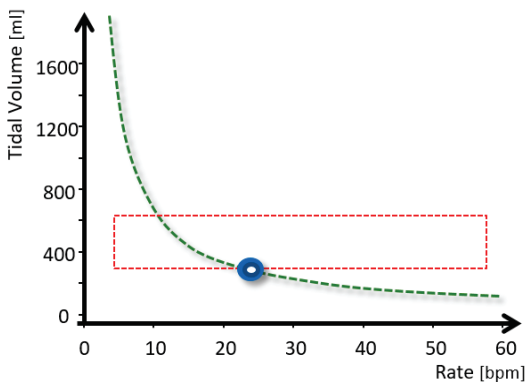

9. Depending on your ventilator, this parameter is named  $P_{limit}$  or  $P_{asvlimit}$ .


Vt is high and RR is very low

### Possible causes:

- ✓ Severe airway obstruction with very long RCexp
- ✓ INTELLiVENT-ASV adapts RR to counteract air-trapping and autoPEEP

### Solutions:

- ✓ Reconsider the diagnosis.
- ✓ Check the ET tube: Verify the size and ensure the positioning is correct with no kinking or biting, and that no secretions are accumulated.
- ✓ If secretions are present, perform suctioning.

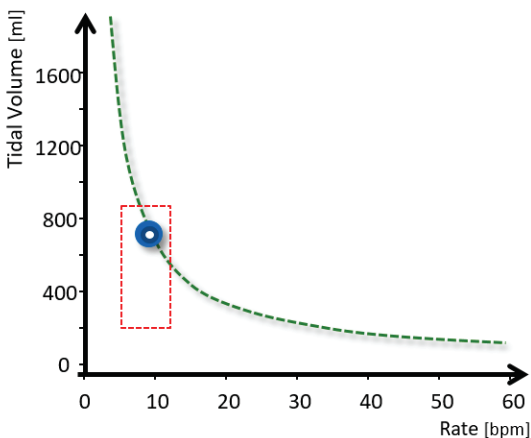





More information:

[www.hamilton-medical.com/intellivent-asv](http://www.hamilton-medical.com/intellivent-asv)



**HAMILTON**  
**MEDICAL**

Manufacturer:

Hamilton Medical AG

Via Crusch 8, 7402 Bonaduz, Switzerland

+41 (0)58 610 10 20

[info@hamilton-medical.com](mailto:info@hamilton-medical.com)

[www.hamilton-medical.com](http://www.hamilton-medical.com)

ELO20220910N.00

The products shown here are not available for purchase by the general public. The information provided here is intended for healthcare professionals only. Always read the labels and follow the product's instructions for use. Specifications are subject to change without notice. Some features are options. Not all features are available in all markets. All images are used for illustrative purposes only and may not accurately represent the product or its use. For all proprietary trademarks (®) and third-party trademarks used by Hamilton Medical AG see [www.hamilton-medical.com/trademarks](http://www.hamilton-medical.com/trademarks). ©2022 Hamilton Medical AG. All rights reserved.

INTELLiVENT-ASV - FAQs and troubleshooting