

PATIENT 132-54/B



- CONTACT
- MAIL
- INFO



**BLOOD RESULTS**

INVESTIGATION	RESULTS	NORMAL VALUE
Haemoglobin	12.0 g/dL	11.5 - 16.0 g/dL
Hematocrit	35.5 %	37.0 - 47.0 %
Red Cell Count	4.23 x 10 <sup>12</sup> /L	4.2 - 5.4 x 10 <sup>12</sup> /L
Haematocrit	35.5 %	37.0 - 47.0 %
Mean Cell Volume	84.0 fL	84.0 - 100.0 fL
Mean Cell Hemoglobin	24.9 pg	26.0 - 32.0 pg
RDW-CV	11.9 %	11.5 - 14.5 %
RDW-SD	14.0 fL	13.0 - 16.0 fL
Platelet Count	145 x 10 <sup>9</sup> /L	130 - 400 x 10 <sup>9</sup> /L
Platelet Hematocrit	1.3 %	0.1 - 0.3 %
Platelet Volume	9.1 fL	8.0 - 11.0 fL
Platelet Count	145 x 10 <sup>9</sup> /L	130 - 400 x 10 <sup>9</sup> /L
Platelet Hematocrit	1.3 %	0.1 - 0.3 %
Platelet Volume	9.1 fL	8.0 - 11.0 fL

**BY DATE**

08/15/13	01:00
08/05	06:00
07/12	08:00
07/01	08:00
06/20	08:00
06/09	08:00
05/28	08:00
05/17	08:00
05/06	08:00
04/25	08:00
04/14	08:00
04/03	08:00
03/22	08:00
03/11	08:00
02/28	08:00
02/17	08:00
02/06	08:00
01/25	08:00
01/14	08:00
01/03	08:00
12/22	08:00
12/11	08:00
11/30	08:00
11/19	08:00
11/08	08:00
10/27	08:00
10/16	08:00
10/05	08:00
09/24	08:00
09/13	08:00
09/02	08:00
08/21	08:00
08/10	08:00
07/29	08:00
07/18	08:00
07/07	08:00
06/26	08:00
06/15	08:00
06/04	08:00
05/23	08:00
05/12	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:00
11/06	08:00
10/25	08:00
10/14	08:00
10/03	08:00
09/22	08:00
09/11	08:00
08/31	08:00
08/20	08:00
08/09	08:00
07/28	08:00
07/17	08:00
07/06	08:00
06/25	08:00
06/14	08:00
06/03	08:00
05/22	08:00
05/11	08:00
05/01	08:00
04/20	08:00
04/09	08:00
03/28	08:00
03/17	08:00
03/06	08:00
02/25	08:00
02/14	08:00
02/03	08:00
01/22	08:00
01/11	08:00
01/01	08:00
12/20	08:00
12/09	08:00
11/28	08:00
11/17	08:0

# Patient monitoring systems

HAMILTON-G5	HAMILTON-S1	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	HAMILTON-HF90	Manufacturer	System	Protocol
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Dräger	Infinity	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	GE Healthcare	Carescape	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Masimo	Patient SafetyNet System and Iris Gateway System	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	Mindray	BeneVision	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Nihon Kohden	BSM-3000K (v08-42 or later)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Nihon Kohden	BSM-1900K (v01-36 or later)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Nihon Kohden	BSM-9101K (v12-06 or later)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Nihon Kohden	BSM-6000K (v02-10 or later)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Nihon Kohden	BSM-1700K (v02-22 or later)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Nihon Kohden	BSM-1500K (v02-22 or later)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Philips	IntelliVue (IntelliBridge)	VueLink Open / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Philips	IntelliVue (VueLink)	VueLink Open
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Spacelabs Healthcare	Ultraview	Polling

- (✓) External device is able to read data from Hamilton Medical device depending on the software version. The accuracy of the displayed monitoring data is not validated by Hamilton Medical.
- x External device is not able to read data / no drivers have been developed.
- o Contact the monitoring system manufacturer for the current status.

# EHR/EMR/PDMS

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	HAMILTON-HF90	Manufacturer	System	Protocol
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Aixelink	ICU Care	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Ascom	Digistat Suite	Polling / Block
(✓)	(✓)	o	o	o	o	o	o	B-Simple	B-ICU.Care	Polling / Block
(✓)	(✓)	o	o	o	o	o	o	Chipsoft	CS-ECIS (Electronic Care Information System)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Clinicomp	EHR	Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Copra System	Copra	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Dedalus	ORBIS	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Dräger	Integrated Care Manager (ICM)	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	GE Healthcare	Centricity Critical Care	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	HIM	QCare	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	iMDsoft	MetaVision	Polling / Block
o	o	(✓)	(✓)	(✓)	(✓)	(✓)	o	IMESO	ICUData	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	LOWTeq	LOWTeq-PDMS intensive care	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Oracle Healthcare	CareAware	Polling / Block
(✓)	(✓)	o	o	(✓)	o	o	o	Philips	ICCA (former ICIP Release D.03)	Polling
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	PICIS	Picis Critical Care Manager	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	TERENUI	Grimoires	Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	zMed Healthcare Technologies	zMed ICU solution	Block

(✓) External device is able to read data from Hamilton Medical device depending on the software version.  
The accuracy of the displayed monitoring data is not validated by Hamilton Medical.

o Contact the monitoring system manufacturer for the current status.

<sup>1</sup> With service pack 1 or higher.

# Connectivity solutions middleware

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	HAMILTON-HF90	Manufacturer	System	Protocol
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	Ascom	Digistat Suite	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Bridge-Tech Medical	Device Connectivity Solution (DCS)	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	Capsule Technologie	Device Driver Interface Library	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Masimo	DeviceConX	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	o	Philips	IntelliBridge	VueLink Open / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	R Group	HealthMosaic/Open Connect	Polling / Block
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	Scion Health Hospital	Pro Touch	Hamilton P2
(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	(✓)	Talis Clinical	Talis-Hub	Block

- (✓) External device is able to read data from Hamilton Medical device depending on the software version. The accuracy of the displayed monitoring data is not validated by Hamilton Medical.
- o Contact the monitoring system manufacturer for the current status.

## Other systems

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	HAMILTON-HF90	Manufacturer	System	Protocol
(✓)	(✓)	(✓)	(✓)	o	(✓)	(✓)	o	EKU	NO-A, EZ-KINOX, CARDINO	Block

- (✓) External device is able to read data from Hamilton Medical device depending on the software version.  
The accuracy of the displayed monitoring data is not validated by Hamilton Medical.
- o Contact the monitoring system manufacturer for the current status.

## Distributed alarm systems

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	HAMILTON-HF90	Manufacturer	System	Protocol
(✓)	(✓)	x	x	x	x	(✓)	x	Ascom	Digistat Suite	Block (ACK)

- (✓) External device is able to read data from Hamilton Medical device depending on the software version.  
The accuracy of the displayed monitoring data is not validated by Hamilton Medical.
- x Not available.



### Your contact

Michel Smits  
Customer Success Manager  
Hamilton Medical

[m-smits@hamilton-medical.com](mailto:m-smits@hamilton-medical.com)